

Soft Contrastor

Function :

Our Soft Contrastors™ are designed to allow upper photos to be taken with a completely black background. This obscures the inside of the mouth and makes the image more presentable for patients, lectures and publications and mask all unsightly elements on the dental photo (cheek, nostrils, mustache, eyelashes, door, etc...). The mat black color increases the contrast of the teeth. They are made of stainless steel sheet coated by a black silicone.

User tips :

The different shapes are according to the views with or without mirror.
They can be easily fold or bent for easier positioning.


Cleaning :

The black mat surface is sensitive. Clean with water and mild soap only. Dry with a sponge.
Avoid washing machine or ultrasonic cleaning device.

Do not leave Soft Contrastors exposed to sunlight or bright light sources to prevent the black silicon from fading.

Sterilization :

Autoclaving is the recommended method of sterilization to 121° (249.8°F) or 134° (273.2°F) for 30 minutes max.). Soft Contrastor must be completely dry before autoclaving or placing in pouches. All other sterilization process are not recommended.

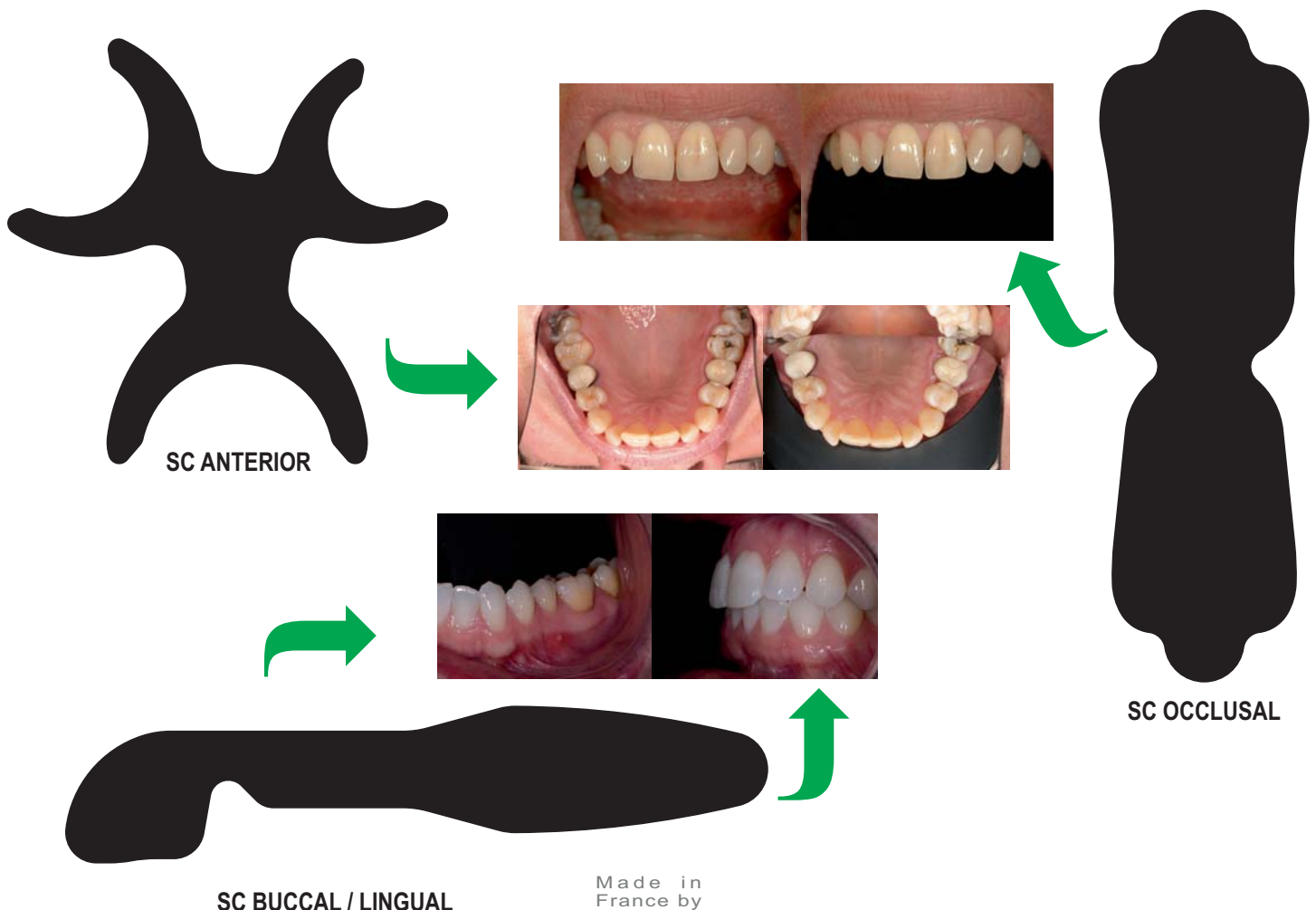

SC BUCCAL / LINGUAL