

# Universal mirror handle

## Function :

With some mirrors fingers may appear on the picture when not properly handled. For the buccal side the mirror is used as a retractor and needs a good grip to have a good strength. That is the reason why the universal handle can be very useful. Universal handle is designed to work with all our mirrors 3mm thickness. Angle between handle and mirror can be adjusted on 35° for better positioning. Unlike other handles rotation can be locked to keep the best position. It can be held either by the practitioner, the assistant or the patient himself.

## User tips :

Universal handle could be positioned anywhere on the mirror according to the use. Mirror angle can be locked by screwing and ensure the good position of the mirror.

## Cleaning :

Handle and head could be full washed separately. Use a soft cleanser compatible with stainless steel. Washing machine or ultrasonic cleaning device are not recommended.

**Before autoclaving, we recommend to take off the mirror and unscrew to avoid a stamp on the ring.**

## Sterilization :

Autoclaving is the recommended method of sterilization to 121° (249.8°F) or 134° (273.2°F) for 30 minutes max.). Handles must be absolutely dry when placed in pouches.

All other sterilization process are not recommended.



Made in  
France by  
**PTJ**  
INTERNATIONAL

CE

